Sterilization of single use products

en - ENGLISH	2	sv - SVENSKA	11
de - DEUTSCH	3	da - DANSK	12
fr - FRANÇAIS	4	cs - ČEŠTINA	13
it - ITALIANO	5	tr - TÜRKÇE	14
es - ESPAÑOL	6	el - ΕΛΛΗΝΙΚΑ	15
pt - PORTUGUÊS	7	bg - БЪЛГАРСКИ	16
no - NORSK	8	hu - MAGYAR	17
et - EESTI KEEL	9	lv - LATVIEŠU VALODA	18
fi - SUOMI	10	lt - LIETUVIŲ KALBA	19

Sterilization of single use products

Basic Principles

Endostops are single-use only and cannot be reprocessed. Endostops are supplied non-sterile and must be sterilized before use. Ensure equipment used is serviced and checked regularly, and manufacturer's instructions are followed.

The user is responsible for the sterility of Endostops and to ensure the following as well:

- Only procedures that are sufficiently validated specifically for devices are used for sterilization.
- The equipment used is regularly maintained, checked and calibrated.
- The instructions regarding the equipment must be respected at all time.
- The user has to be trained adequately.
- In addition to these instructions, please observe the legal regulations valid in your country as well as the hygiene regulations of the dental practice.

Packaging

Packaging can influence the attainment of sterilization conditions. We recommend the use of sterilization pouches/ reels compliant with ISO 11607-1 and suitable for steam sterilization. The packaging shall be large enough to avoid stressing the sealing steam.

Warning! After the heat sealing process, the sealing seam shall be checked visually for any defects. In case of defects, the packaging must be opened and the product repacked and sealed

Sterilization

Sterilize Endostops only in a steam autoclave using distilled water and at the temperature specified on the relevant symbol. Do not use chemical, cold sterilization or dry heat sterilization.

Steam sterilization shall be validated according to ISO 17665 series. The autoclave shall be compliant with EN 13060.

- Follow carefully the instructions for use of the autoclave.
- Inspect the autoclave regularly according to the instructions for use. Service and clean the device frequently, according to the instructions for use.
- Ensure that maximum load is not exceeded.
- Place all components in a sterilization disposable pouch (refer to "Packaging").
- Endostops shall not touch the autoclave walls, since the temperature there might be higher.
- We recommended to use always the following cycle (Prion Cycle):

Temperature of sterilisation: 134°C

Time of sterilisation: 20 min

Drying time: 20 min

- Remove Endostops from the autoclave immediately after sterilisation.
- Check the integrity of packaging and products.

Storage

- Store sterilized product in dry and clean conditions at ambient temperature.
- Keep product sealed in the sterilization pouch until ready to use.

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