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Agente co-formativo de pasta de hidróxido de cálcio para curativos temporários de canais radiculares.

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DIRECTIONS FOR USE **EN**

Calcium hydroxide Powder

Co-formative agent of calcium hydroxide paste for temporary root canal dressing.

Composition Calcium Hydroxide 100%.

Properties

When mixed with distilled water, Calcium hydroxide powder forms a paste of creamy consistency with a pH above 12.

Indications Temporary root canal dressing for endodontic therapy.

Directions for use

Meticulously prepare, clean and dry the root canal.

Mix extemporarily calcium hydroxide powder with distilled water to reach a paste of creamy consistency.

Fill the canal with the paste using a Lentulo spiral or equivalent instrument. Do not overfill the root canal system to avoid apical extrusion.

Fill the cavity with a temporary restorative filling material for a tight sealing during intersession to avoid contamination of the root canal system.

Remove completely the paste at the following appointment. Calcium Hydroxide Paste is easily removed with an endodontic file and a few drops of saline solution.

Precautions

Adjust the volume of paste in the apical area according to the anatomical shape of the canal to prevent the risk of overfilling.

As temporary root canal dressing, the maximum duration of treatment should be 30 days.

Contraindication No known or foreseen undesirable effect when used as intended.

Warning

This product is made of calcium hydroxide, a skin irritant which can cause serious damage to the eye. Avoid contact with eyes, oral mucosa and skin to prevent possible irritation or injury. In accordance with best clinical practice, the use of rubber dam and the wearing of safety glasses and a waterproof bib by the patient are recommended. Practitioners should wear protective gloves, nose and mouth protection and safety goggles.

In case of contact with the skin or oral mucosa, immediately remove the product from the tissue and rinse thoroughly with water. If skin or mucous membrane irritation develops, get medical advice/attention.

In case of contact with eyes, rinse immediately with plenty of water for several minutes, holding the eyelid open. Call a physician immediately.

If swallowed, rinse mouth with water and drink plenty of water. Get medical advice/attention. Keep the exposed person at rest. Do not induce vomiting.

Storage Store at room temperature between 5°C and 25°C, protected from direct light.

Shelf life 5 years. Do not use the product after the expiry date.

Disposal Discard the empty jar as per local regulation.

Content of packaging

Jar of 50g

For dental professional use. Made in Switzerland.

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