




SMART PRODUCTS
FOR ENDO LOVERS



Irriflex[®]

Quick User Guide

30G flexible root canal
irrigation needle with
2 back-to-back side-vents

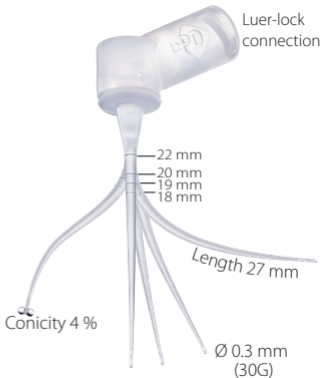



- Use the markers or a **stopper** to premeasure **IrriFlex®** 1-2mm short of the working length.
- When indicated, gently bend **IrriFlex®**.

1. Irrigate the pulp chamber, avoid contacting the pulp chamber floor.
2. Insert **IrriFlex®** into the canal, ensure it is passive and apply gentle pressure on the plunger.

3. While irrigating move **IrriFlex®** up and down (1-2mm) to circulate the irrigant. keep 1-2mm short of the working length.

