DIRECTIONS FOR USE	GB
PD MTA White	
Bioceramic repair material	

Composition
Tricalcium silicate, Dicalcium silicate, Tricalcium aluminate, Calcium oxide, Calcium Tungstate.

Proprietés
PD MTA White is a radiopaque root canal repair material material. The material composed of several mineral oxides is made up of thin hydrophilic particles showing the following advantages:
– Optimized particle size allowing homogeneous and complete wetting during mixing
– Avoids bacterial migration and penetration of tissue fluids into the root canal thanks to its excellent marginal sealing capacity
– No need to dry the treated site. PD MTA White properties are not decreased in a moist environment
The powder mixed with distilled water forms a gel, which cures in a humid environment. The material starts setting after approximately 10 minutes and the final setting time is 15 minutes. It is not necessary to wait for the final setting to continue the treatment procedure.

Indications
– Treatment of perforations of root canal and furcation
– Treatment of internal resorption
– Retrograde root canal filling after apical resection
– Pulp capping
– Pulpotomy
– Apical plug during apexification treatment

Directions for use
Place on a glass slab the content of 1 sachet of PD MTA White and add 1 drop of distilled water next to the Powder. Mix PD MTA White by gradually bringing the liquid into the powder and mix it evenly for 30 seconds. The mixture must show a creamy consistency. Place the mixture immediately on the selected site with an appropriate applicator (We do recommend the MAP system). After application, condense the material with a condenser or absorbent paper point moistened with distilled water.

Warning and precaution
Use PD MTA White immediately after mixture. The product dehydrates very quickly and must be discarded if this happens. Clean the applicator immediately after the placement of the material. If the material hardens inside the applicator it cannot be removed anymore, thus making the instrument inoperative. Do not use PD MTA White to fill a complete root canal. Do not use PD MTA White in contact with the gingival sulcus. This would lead to complete dissolution of the material. Acidic pH (lesions) prevents the setting reaction of PD MTA White. Avoid apical extrusion of the material. The successful use of MTA depends on a correct diagnosis, the operative technique, the condition of the treated tooth and the general health of the patient.

When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitives to any of these ingredients.

Storage
PD MTA White must be stored in its original packaging at room temperature. Protect from humidity.

Shelf-life 3 years.	
Packaging <p>Kit with 4 sachets of 140mg powder and 1 bottle of 3 ml distilled water.</p>	 REF 20190

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