DIRECTIONS FOR USE	EN
Opacal <p>Calcium Hydroxide Paste</p>	

Radiopaque Calcium Hydroxide paste for temporary root canal dressing.

Composition
Calcium Hydroxide: 30%, Barium Sulfate 24%, excipient.

Properties
Opacal is a radiopaque, creamy and ready-to-use calcium hydroxide paste with a pH above 12. Paste delivery in root canal is ensured by dedicated application tip.

Indications
Temporary root canal dressing for endodontic therapy.

Directions for use

- Meticulously prepare, clean and dry the root canal.
- Prepare the syringe by clipping the finger grip onto the syringe wings.
- Place a new application tip on the luer–lock after uncapping the syringe.
- Fill the canal with Opacal from apical portion of root outward. Use the enclosed tip and stay at least 2 mm shy of apex.
- Fill the cavity with a temporary restorative filling material for a tight sealing during interession to avoid contamination of the root canal system.
- Check the quality of the filling by radiographic inspection.
- Discard the application tip and immediately close the syringe with the original cap to avoid dryness of Opacal in the syringe. Disinfect the syringe.

Remove the paste completely at the following appointment. Opacal is easily removed with an endodontic file and a few drops of saline solution.

Precautions
Use enclosed tip and stay at least 2mm shy of apex to limit risk of apical extrusion. Adjust the volume of paste in the apical area according to the anatomical shape of the canal to prevent risk of overfilling. Each application tip is single use. Discard it after each treatment and use a new one for each patient. Disinfect syringe between each patient to avoid potential cross-contamination. Test paste flow from syringe by expulsing a small amount onto a mixing pad before using intraorally with tip. Never force syringe plunger to avoid uncontrolled splash of product into the oral cavity. If the application tip is blocked, discard it and use a new one. As temporary root canal dressing, the maximum duration of treatment should be 30 days.

Contraindication
No known or foreseen undesirable effect when used as intended.

Warning
This product is made of calcium hydroxide, a skin irritant which can cause serious damage to the eye. Avoid contact with eyes, oral mucosa and skin to prevent possible irritation or injury. In accordance with best clinical practice, the use of rubber dam and the wearing of safety glasses and a waterproof bib by the patient are recommended. Practitioners should wear protective gloves, nose and mouth protection and safety goggles. In case of contact with the skin or oral mucosa, immediately remove the product from the tissue and rinse thoroughly with water. If skin or mucous membrane irritation develops, get medical advice/attention. In case of contact with eyes, rinse immediately with plenty of water for several minutes, holding the eyelid open. Call a physician immediately. If swallowed, rinse mouth with water and drink plenty of water. Get medical advice/attention. Keep the exposed person at rest. Do not induce vomiting.

Storage
Store at room temperature between 15°C and 25°C, protected from direct light.

Shelf life
3 years. Do not use the product after the expiry date.

Disposal
Discard each tip, and the syringe when fully used, as per local regulation.

Packaging Description	 REF
1 syringe of 3 ml + 10 single use application tips + 1 finger grip	11020
<i>Additional Tips can be ordered with dedicated reference:</i>	
10 single use application tips	11022

For dental professional use. Made in Switzerland.

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