

# P-One Dispenser for Cotton pellets

Innovative device  
with automatic  
expulsion system



## **P-One**

Dispenser for Cotton pellets

Innovative design and user-friendly functionality

Automatic gravity-based expulsion system

High chemical resistance materials

Simple to use and easy to disinfect

100% designed and manufactured in Switzerland




# P-One Dispenser for Cotton pellets

## Technical features

- POM plastic based
- Stainless steel pressure ring
- 2-position closing cap
- 2 grids available:
  - large grid for pellets N°1 and 2
  - small grid for pellets N°3 to 5
- Bulk refill method
- 15 ml (3 g) of cotton capacity



## QUICK USER GUIDE

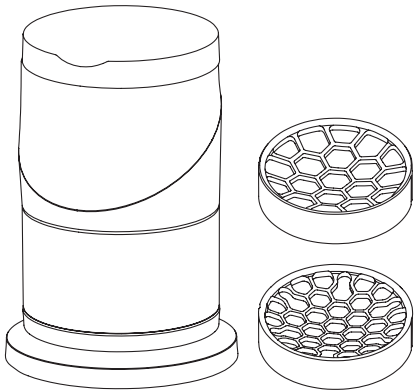

### P-One Dispenser for Cotton pellets REF 33280

- 1 dispenser
- 1 large grid for pellets N° 1 and 2
- 1 small grid for pellets N° 3, 4 and 5

1. Separate foot part from pellets collecting tube and turn it upside down
2. Choose appropriate grid and slide it into collecting tube
3. Insert pellets inside collecting tube up to limit mark
4. Insert foot part inside collecting tube
5. Turn complete device back in place, P-One is ready to use!

