




SMART PRODUCTS  
FOR ENDO LOVERS

# IrriFlex<sup>®</sup>

## Quick User Guide


Single use 30G flexible root canal irrigation  
needle with 2 back-to-back side-vents

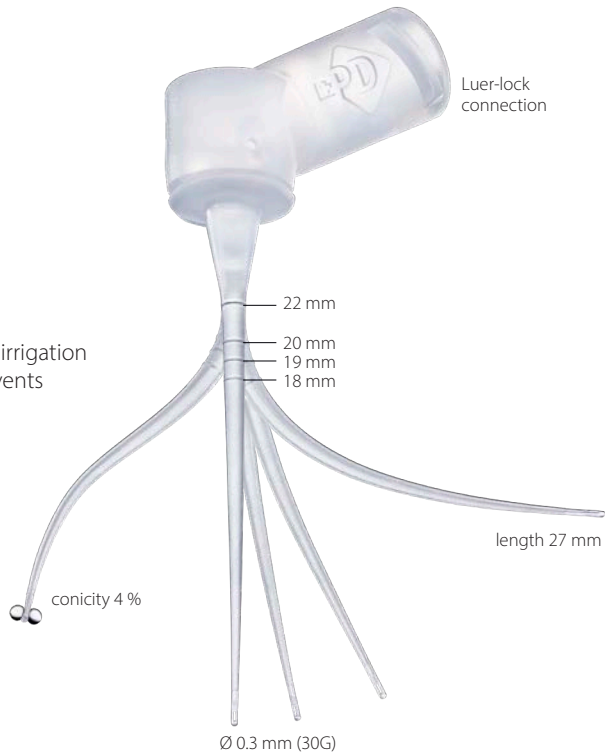

# Setting Instructions



1. Open the packaging by pulling along pre-cut notch provided for this purpose.
2. Prefill your Luer-lock syringe with one of the compatible fluids: (heated) stabilized NaOCl up to 6 %, EDTA 17 %, Chlorhexidine digluconate 2 %, Acid citric up to 40 %, distilled water.

 **Note:** Dual Rinse® HEDP is also a compatible solution.



3. Insert the connector part of the needle into the male nose of the previously filled Luer-lock syringe and lock the needle by rotating it clockwise.

 **Note:** Make sure the needle is correctly locked on the syringe.



4. Identify the 4 depth marks on the needle (18 mm, 19 mm, 20 mm and 22 mm) to respect your working length.

 **Note:** A stopper may be used to indicate the working length.

**First time user:** Before use, check the pressure needed on the syringe barrel to create two balanced jets exiting the needle. This step also allows you to control the flowability of your irrigation solution.

# Irrigation



**Note:** If needed, pre-bend the tip to access easily into the canal.



5. Start irrigating in the pulp chamber and gently insert the needle into the canal orifice.



**Note:** Avoid hitting the pulp chamber floor with the tip of the needle in order to prevent any damages to the back-to-back vents.



6. The tip of the needle easily slides down into the canal. Make sure to respect your working length.



**Note:** Make also sure that the needle can move freely to ensure constant flowback of the irrigation solution.



**SMART PRODUCTS  
FOR ENDO LOVERS**

## Product references

	Ref.	Suggested retail price
BOX OF 20 PIECES	21400	37.80 CHF   €
BOX OF 40 PIECES	21401	63.00 CHF   €



[pd-irriflex.com](https://pd-irriflex.com)

This Quick User Guide is a simplified version of the Instructions For Use (IFU) aiming to highlight and summarize the key use points. Please always refer to the official IFU provided with the physical products or on the website. Only the IFU is authoritative.



Rue des Bosquets 18  
1800 Vevey  
Switzerland



**Produits Dentaires SA**  
Rue des Bosquets 18  
1800 Vevey . Switzerland



+41 21 921 26 31



[info@pd-dental.com](mailto:info@pd-dental.com)



1639