


EssenSeal®

Quick User Guide


1

Preparation phase

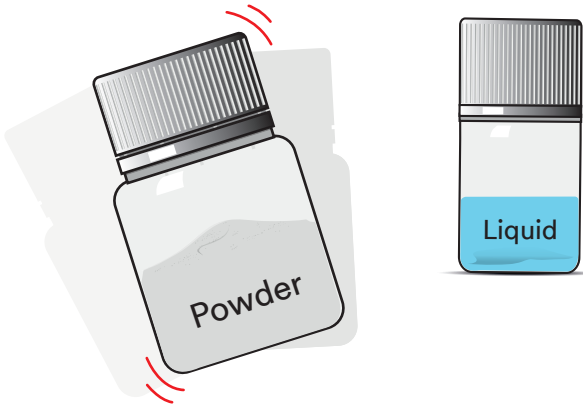

Shake the bottle before use to aerate the powder.

2

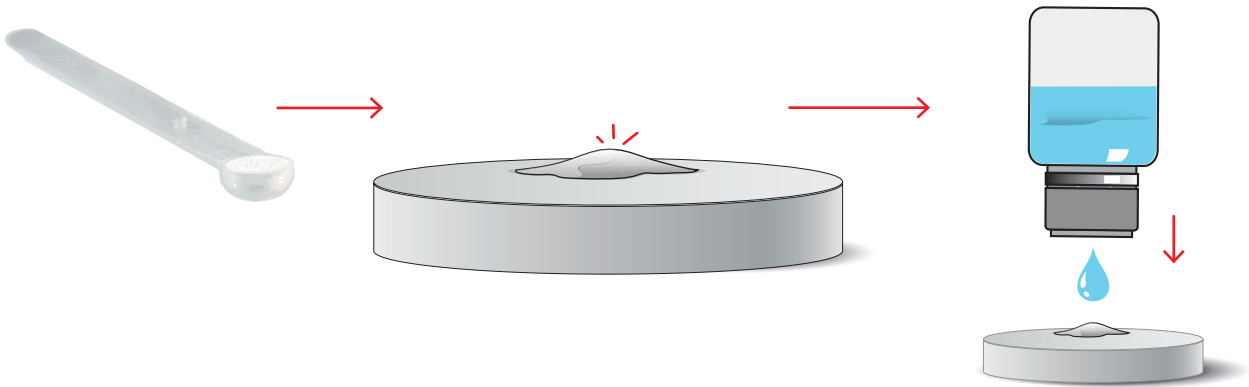

Fill a level spoonful with powder.
On a smooth glass mixing block, **place 1 level dosing spoon of powder and 1 drop of liquid.**
A reminder of the ideal mixing ratio is placed on the vials.

Caution : Hold the bottle upside down and wait until a single drop falls onto the mixing plate.

3

A

Right after use!

A **Close** powder and liquid vials immediately after use.


B

B With a spatula, **prepare** the root canal sealer extemporaneously by gradually incorporating the powder into the liquid.
The mixture should have a **smooth and shiny appearance.** Consistency should be creamy.

4

Treatment phase

■■■■■■■■■■ Treatment in process ■■■■■■■■■■

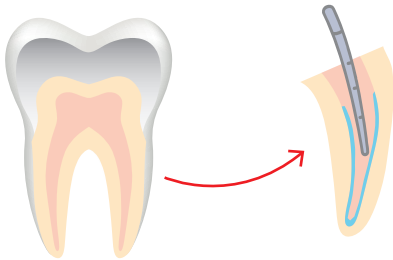

Apply the sealer to the **canal walls** using a root filler, gutta-percha points or any other device suitable for this purpose.
The choice of filling technique is left entirely to the practitioner.

5

The working time at temperatures **below 25°C** is more than **30 minutes.**
The setting time in the mouth (37°C and > 95 % relative humidity) is less than **3 hours.**

Note : Heat and humidity are known to accelerate the setting of the cement.

Check the quality of the filling by **radiographic inspection.**
In case of re-treatment, use conventional chemo-mechanical filling removal techniques.



SMART PRODUCTS
FOR ENDO LOVERS