DIRECTIONS FOR USE	EN
--------------------	----

Plastic pipettes

Intended use

Device for delivering small quantities of liquid into root canal or pulp chamber during endodontic or restorative treatment.

Properties

Flexible and translucent plastic pipette for the delivering of small quantities of liquid (drop by drop).

The different color-coded rings allow to identify the liquid contained in each pipette if several of them are used simultaneously.

Direction for Use

The liquid to be delivered is sucked by the pump part of the pipette previously pressurized manually. The liquid is then delivered to the root canal anatomy / pulp chamber at the outlet of the pipette's fine tip. The dosing/expelling of the liquid is done by generating pressure on the pump part of the pipette.

Compatible delivery solutions

The components of the pipette have good compatibility properties with a large range of liquids. The following liquids have been validated for safe use with this pipette: orange oil solvent, eucalyptol solvent, EDTA 17%, Sodium hypochlorite up to 6%, chlorhexidine 2%, distilled water.

Adverse reactions and contraindications

None known.

Precautions and warning

The plastic pipette is a single-use device. Do not reuse it in order to avoid cross-contamination between different patients / liquid or degradation of the product.

Do not disconnect the individual parts of the pipette. Do not use in case of leakage.

Orange oil solvent and eucalyptol solvent should not be stored in the pipette for longer than 10 minutes.

Do not sterilise before use. There is no need of disinfection before use.

Storage

Keep away from direct light. Store at room temperature.

Shelf life

5 years. Do not use the product after the expiry date.

Disposal

Discard each pipette as per local regulation.

Packaging description

	[REF
Box of 5 plastic pipettes 0.3 ml	11750
Box of 25 plastic pipettes 0.3 ml	11754
Box of 5 plastic pipettes 1 ml	11752
Box of 25 plastic pipettes 1 ml	11757
For dental professional use. Made in Switzerland.	

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