Vaseline or similar products may be used for smooth working and to facilitate the motion of the screw parts. Matrix bands are single use devices. Do not reuse.

Contraindication
None known.

Self-life
The matrix retainer can be used as long as it is functional (without sign of degradation).

Disposal
Dispose the product as per local regulations.

Packaging description
Matrix Retainers Junior
Matrix Retainers Junior + 12 Matrix Bands Assortment
Matrix Retainers Senior
Matrix Retainers Senior + 12 Matrix Bands Assortment
Matrix Retainers Universal
Matrix Retainers Universal + 12 Matrix Bands Assortment
For dental professional use. Made in Switzerland.

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Reprocessing protocol

The Tofflemire matrix retainer secures the appropriate Tofflemire matrix band in place. The whole matrix system enables to perform restoration with anatomical proximal contours and contact areas, by creating a temporary interproximal surface.

Properties

The Tofflemire matrix retainer can be used lingually and buccally. It has a non-slip grip handle and polished surfaces.

The Tofflemire matrix retainer exists in 3 variants: Junior, Senior and Universal.

Direction for Use

Q. How to use the matrix retainer before first use (see protocol below).

1. Prior to placing the matrix band, place a wedge in the gingival embrasure to check the bond between the occlusal surface of the wedge and the gingival caovo-surface margin. Wedges help to compress the matrix band against the tooth structure to create a tighter seal, preventing gingival overhang. It also provides slight separation between the wedge and the gingival band to help to create a tight seal around the adjacent tooth.

2. Choose appropriate matrix band. If necessary, the matrix band can be burned or shaped.

3. Place the matrix band into the matrix retainer as follows:

a) Turn the long knob to move the locking screw down the head of the retainer.

b) Turn the short knob to loosen the retaining screw (the retaining screw should disengage from the sliding slot).

c) Fold the band to form a loop. Note that when the band is folded, one edge has a smaller circumference than the other, this design accommodates the difference in tooth circumferences at the contact and gingival levels. There are 2 main rules for correct placement of the matrix system on the tooth:

- The open end of the retainer head must always be placed toward the gingiva, and

- The outer circumference of the band must always be placed toward the gingiva.

4. Place the matrix band into the matrix retainer as follows:

a) Turn the long knob to move the locking screw down the head of the retainer.

b) Turn the short knob to loosen the retaining screw (the retaining screw should disengage from the slot).

c) Fold the band to form a loop. Note that when the band is folded, one edge has a smaller circumference than the other, this design accommodates the difference in tooth circumferences at the contact and gingival levels. There are 2 main rules for correct placement of the matrix system on the tooth:

- The open end of the retainer head must always be placed toward the gingiva, and

- The outer circumference of the band must always be placed toward the gingiva.

5. Tighten the matrix band around the tooth by turning the long knob. Tighten the matrix as much as possible around the tooth. This allows for proper restorative material condensation and contour development. Place the wedge, once the band is tight in the gingival embrasure, loosen the matrix band slightly by turning the knob 1/4 of a turn.

6. Perform the restorative treatment.

7. Once the restorative treatment is completed, remove the retainer, by moving counter clockwise the small knob to free the band from the retainer. Then remove the band. Carefully remove band from each contact point.

8. Clean and disinfect the matrix retainer as described below.

Cleaning and Disinfection

For hygiene and safety purposes, the matrix retainer must be cleaned and disinfected before each use. Do not use to clean and disinfect.

General recommendations

Use only a disinfecting solution which is approved for its efficacy (VAH/DGHM-listing, CE marking, FDA approval) and in accordance with the instructions of the disinfecting solution manufacturer. It is recommended to use anticorrosion disinfecting and cleaning agents (like: Alconox™ Liquinox™ Critical Cleaning Liquid Detergent).

For your own safety, please wear protective equipment (gloves, glasses). The user is responsible for the correct use of the instrument for the first time and for further usage as well as for the usage of damaged or dirty instruments otherwise applicable after cleaning.

Limitations and restrictions on reprocessing

The matrix retainer can be used as long as it is functional (without sign of degradation). The appearance of defects such as cracks, deformations (bent, twisted), corrosion, loss of marking, are indications that the device is not able to fulfill the intended use with the required safety. The water quality has to comply with the local regulations especially for the last rinsing step or with a washer-disinfectant. Do not use acid (pH<6) or alkaline (pH>8) solutions.

Reprocessing protocol

1. Disassemble the device. Separate the two knobs from the head of the matrix retainer.

2. Soak all instruments in an alcohol-free detergent and disinfecting solution per the manufacturer's recommended instructions. The detergent should contain di- or triethanolamine as corrosion inhibitor (see general recommendation).

3. Do not use disinfecting solutions containing Phenol or any products which are not compatible with the instrument (see general recommendation).

4. If visible impurities are observed on instruments, brush the surface using a soft-bristled brush for maximum effect.

5. If a pre-desinfectant solution contains a corrosion inhibitor, it is recommended to rinse the instrument just before the cleaning.

6. Place the devices in a kit or container to avoid any contact between them.

7. Prepare a solution of alcohol-free detergent per the manufacturer's recommended instructions. The detergent should contain di- or triethanolamine as corrosion inhibitor (see general recommendation).

8. Completely submerge the devices in the cleaning solution and sonicate for a minimum of 5 minutes.

9. Allow the device to air dry on a non-woven paper with filtered compressed air.

10. Reassemble the device.

11. Rinse the device with a clean environment.

Precautions

Do not force the device at any time. The blocking of the device may be due to the presence of foreign object. If this is the case, remove it before proceeding.

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Tofflemire Matrix Retainers

Intended use

The Tofflemire matrix retainer secures the appropriate Tofflemire matrix band in place. The whole matrix system

enables to perform restoration with anatomical proximal contours and contact areas, by creating a temporary

interproximal surface.

Properties

The Tofflemire matrix retainer can be used lingually and buccally. It has a non-slip grip handle and polished

surfaces.

The Tofflemire matrix retainer exists in 3 variants: Junior, Senior and Universal.

Direction for Use

Q. How to use the matrix retainer before first use (see protocol below).

1. Prior to placing the matrix band, place a wedge in the gingival embrasure to check the bond between the

occlusal surface of the wedge and the gingival caovo-surface margin. Wedges help to compress the

matrix band against the tooth structure to create a tighter seal, preventing gingival overhang. It also pro-

vides slight separation between the wedge and the gingival band to help to create a tight seal around the

adjacent tooth.

2. Choose appropriate matrix band. If necessary, the matrix band can be burned or shaped.

3. Place the matrix band into the matrix retainer as follows:

a) Turn the long knob to move the locking screw down the head of the retainer.

b) Turn the short knob to loosen the retaining screw (the retaining screw should disengage from the slot).

c) Fold the band to form a loop. Note that when the band is folded, one edge has a smaller circumference than the other, this design accommodates the difference in tooth circumferences at the contact and gingival

levels. There are 2 main rules for correct placement of the matrix system on the tooth:

- The open end of the retainer head must always be placed toward the gingiva, and

- The outer circumference of the band must always be placed toward the gingiva.

4. Tighten the matrix band around the tooth by turning the long knob. Tighten the matrix as much as pos-

sible around the tooth. This allows for proper restorative material condensation and contour develop-

ment. Place the wedge, once the band is tight in the gingival embrasure, loosen the matrix band slightly by

turning the knob 1/4 of a turn.

5. Perform the restorative treatment.

6. Once the restorative treatment is completed, remove the retainer, by moving counter clockwise the small knob to free the band from the retainer. Then remove the band. Carefully remove band from each contact point.

7. Clean and disinfect the matrix retainer as described below.

Cleaning and Disinfection

For hygiene and safety purposes, the matrix retainer must be cleaned and disinfected before each use.

Do not use to clean and disinfect.

General recommendations

Use only a disinfecting solution which is approved for its efficacy (VAH/DGHM-listed, CE-marking, FDA approval)

and in accordance with the instructions of the disinfecting solution manufacturer. It is recommended to use

anticorrosion disinfecting and cleaning agents (like: Alconox™ Liquinox™ Critical Cleaning Liquid Detergent).

For your own safety, please wear protective equipment (gloves, glasses). The user is responsible

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