Eugenate Desobturator

Solution for softening oxy-eugenolates and Gutta Percha based obturating materials into the root canal.

Composition Oil of orange

Properties

Eugenate Desobturator is a solution to be used specifically for the softening and subsequent mechanical removal of eugenate type and Gutta Percha based obturating materials from the root canal.

Indications

Desobturation of fillings of the eugenate type and Gutta Percha based materials from the root canal.

Directions for use

- ▶ After mechanical cleaning of the pulp chamber, place a drop of the solution inside the root canal with either a cotton pellet or a plastic pipette
- ▶ Allow approximately 1 minute for the solution to act and then desobturate with a broach dipped in the solution. Repeat the operation each time resistance is felt until the root canal is clear to the apex
- ▶ Rinse, clean and dry thoroughly after desobturation and prior to inserting a new filling

Warning

When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of this ingredient.

Storage

Store at room temperature between 5°C and 30°C, protected from direct light.

Packaging

Bottle of 15 ml

REF

11555

For dental professional use. Made in Switzerland.

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