# Rosen Root Canal Cement silver free

Radiopaque Root canal sealing cement

## Composition

**Powder** Zinc Oxide 41.2%, Rosin 16%, Thymol Iodide 12.8%, Radiopaque excipient ad 100 %

**Liquid** Oil of Cloves 78%, Canada Balsam 22%

## Indications

- ▶ Permanent root canal sealing

## Directions for use

- ▶ Spatulate the liquid and the powder to obtain a paste of creamy consistency. The recommended powder/liquid (P/L) weight ratio is between 1.9 and 3.0 (g/g)
- ▶ Clean the root canal and air-dry thoroughly
- ▶ Work the paste into the canal with a probe or a rotary paste filler
- ▶ Avoid air pockets and extrusion beyond the apex
- ▶ The setting time (tested according to ISO 6876) is 70-80 minutes, depending on the P/L ratio (the thicker the paste, the faster is the setting time)
- ▶ Can be used with any type of endodontic points

## Warning

Due to the presence of eugenol, avoid using cement with resin-based fillings.

When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

## Storage

Store at room temperature between 5°C and 30°C, protected from direct light and humidity.

## Packaging

Portion of 15 g powder and 15 ml liquid

REF

10245

For dental professional use. Made in Switzerland.

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© PD 170508